N I C A L _____
SMART MICRO ST 

Manuale d'uso
Short user manual

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Choose one of the six plate sizes (0, 1, 2, 3, 4, 5) to make the radiological exam, being careful to handle the plate with the use of clean gloves. Use a cardboard light cover of the same size to protect the surface of the plate during handling. The cardboard cover has the dual task of blocking ambient light from hitting the plate after the exposure and to protect it from fingerprints & wear during handling. Place the plate inside the light cover by matching the blue part of the plate with that of the cover (Fig. A.1-A.2). Once together, both the plate and light cover are ready to be placed into the hygienic bag. The black side of the light cover is the side which will be exposed to X-rays ("Tube Side"). The "Tube Side" of the cover should be inserted into the hygienic bag so as to appear through the transparent window (Fig. A.3). Avoid exposing the transparent side of the bag to unnecessary ambient light after the X-Ray exposure as this can bleed through to the film and degrade the image.

Suggestions of X-ray tube settings:

The NSM ST VET phosphor plates are very sensitive so the exposure parameters of the X-Ray generator should be set carefully. The time of exposure should not exceed 250 ms. Time is, however, dependent on the RX intraoral tube type being used, parameters such as mAs and kV, the distance between the tube and the plate and the animal anatomy. The system should be tested using extracted teeth or a dental phantom to determine the optimal exposure settings for your system configuration. Intraoral film holders should be used to aid in the correct angulation & positioning of the tube head thereby ensuring the best possible diagnostic images.

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Operation of the Nical Image Plus VET Software

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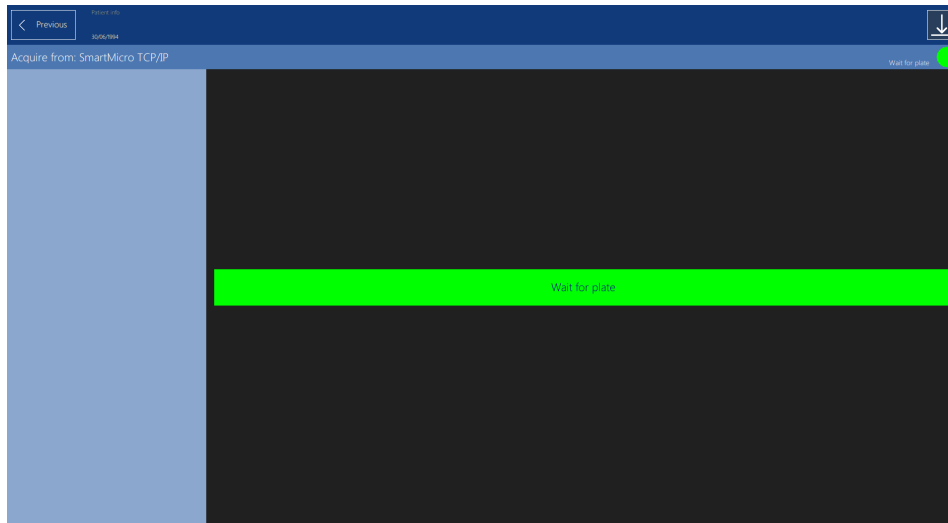
Check that the external power supply is correctly plugged into the power socket at the rear of the Scanner. Connect the scanner to the PC using the USB or Ethernet cable; the machine will automatically turn on and will be in "ready" mode. Note: there is no power switch on the scanner.

Ensure that the USB dongle is plugged into the PC and then click on the Nical Image Plus VET icon to start the software. Enter the password to access the patient file screen, select the correct patient for the radiological examination and click "Intraoral". The new software Nical Image Plus VET allows two different connection modalities, through Ethernet or USB interface. After choosing the desired connection modality, a voice guidance and the instructions on the screen will help you in the image acquisition, making the process simple and fast (Fig. B.1). Once the acquisition is terminated, it will be possible to process the image by adding new and advanced filters, objects (measures, etc.) and by modifying contrast and brightness.

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Introduction of the phosphor plate to the Nical Smart Micro ST VET machine:

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Insert the exposed X-ray plate into the machine by following these instructions:

1. *Remove both plate and light cover from the hygienic bag (Fig. C.1);*
2. *Place the plate with light cover into the slot on the front of the scanner with the black "Tube Side" towards the front left of the machine. Ensure that the light cover and film are positioned flatly on the insertion 'tray' and push them gently into the scanner until the limit stop is hit (Fig. C.2); at this point, gently release the grip on the light cover and remove it from the machine leaving the plate on the tray at the opening. The removal of the light cover will trigger the automated insertion of the plate into the scanner for the acquisition of the diagnostic image (Fig. C.3);*
3. *At the end of the reading process, the plate will be returned to the entrance of the scanner ready for collection by the operator. Using the light cover (to avoid touching the plate with your fingers), collect the plate and insert them into a fresh hygienic bag ready for the next X-Ray. (Fig. C.4).*

Fig. C.1

Fig. C.2

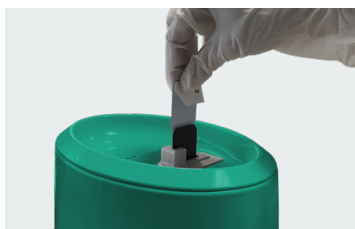
Fig. C.3

Fig. C.4